

E120-2 User Guide

1 Using the Joystick

To make the lift move: Simply push and hold the joystick in the direction you wish to travel. After a slight delay, the Stairlift will begin to move until you release the joystick or reach the top/bottom of the Rail.

2 Using Your Remotes

You can also use your call and send units to move your Stairlift. Press the top red button on the remote to move upwards and the bottom black button to move the Stairlift down the stairs.

3 Retractable Seatbelt

Simply grab the retractable side and clip into the locking side with the red button. Once the lift has stopped and you are ready to get off, hold onto the belt and simply press the red button to release. The belt will then retract back out of the way.

Must be worn at all times


4 Swivel Seat

Your Stairlift comes equipped with a swivel lever as standard. Once you reach the top of the stairs pull the lever upwards. This will allow the seat to swivel out so you can easily step off. To swivel back simply lift the lever again and swivel back to the position, once you let go the lever will safely slot into position.

If you have an upgraded Powered Swivel. Simply push the joystick when arrived at the top to swivel. Either release the joystick to stop or wait for it to reach its limit. Don't worry the lift will not allow you to swivel too far. To swivel back simply push the joystick across to the downward direction.

5 Safety Arm

For added safety the Stairlift is fitted with a Safety Arm. The arm on the downside must be in its down position to be able to move.

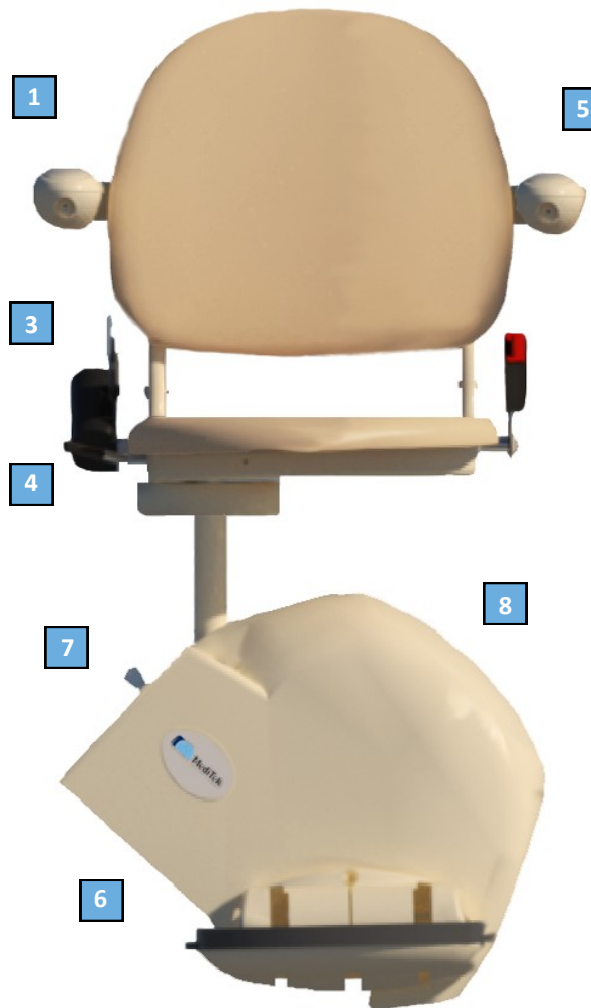

6 Safety Edges

If your Stairlift comes into contact with any objects on its travel, the Stairlift will stop. The lift will not be able to move until the object is out of the way.

7 Upper Chassis Controls

Located on the upper part of the Stairlift's Chassis is: the ICS Display, Key Switch and the Power ON/OFF Switch.

ICS Display—This display tells you in Real-time what the Stairlift is doing. Please refer to the installation guide for full explanations.

All you need to know is:

LIFT READY—the lift is ready to use

MOVING UP—the lift is moving upwards

MOVING DOWN—the lift is moving downwards

ARRIVED TOP—the lift has arrived at the top

ARRIVED BOTTOM—the lift has arrived at the bottom

SEAT SWIVEL—the seat has fully swivelled

SERVICE—the Stairlift is ready for a service

Key Switch—to make the Stairlift operational, the key must be inserted and turned to the ON position. To disable the Stairlift simply turn to the OFF position. The Key can now also be removed for added security.

Power ON/OFF Switch—the Switch turns your Stairlift ON and OFF. This is linked to the batteries and we recommend you leave it ON, if the lift is OFF, the batteries will not charge. A green band will show on the switch when switched ON. This can be used in an emergency to immediately stop the lift.

8 Power Cut?

In the event of a power cut or if the Stairlift was to stop on the during travel, the Stairlift can be wound down using the Hand winder tool provided.

On the main Chassis:

1. Turn OFF using the Power ON/OFF Switch

2. Remove the white plug

3. Insert the Hand Winder onto the shaft

4. Turn clockwise to move Up and anti-clockwise to move Down.